Wheelchair Lift Operational Checklist

PRE-OPERATION CHECKS			OPERATION CHECKS		
	Visually check Lift for bent or broken parts, or for hydraulic fluid around base. Have hydraulic fluid leaks, and the fluid level in reservoir, checked by a qualified technician.		Operate the Lift through one full cycle to do the following checks. Use Lift operating instructions to RAISE and LOWER platform.		
	Check that all Decals are in place, undamaged, and legible		Make sure the Lift responds properly to hand control switches.		
	Check that all anti-slip and safety striping is in place	_	_		
	If the Lift is dirty, clean it.	Ц	Listen for unusual noises while the Lift operates.		
	Make sure the ON/OFF switch on the Lift Cover is turned ON. If indicator on the switch is illuminated, Lift has power to operate.		Watch for uneven movement of the Lift arms, platform, and inboard and outboard rollstops.		
			NOTE: If any of the OPERATION CHECKS have incorrect indications, discontinue operat-		
	Perform check WEEKLY		ing Lift until repairs are performed by a qualified technician.		

Date	Initials	Time Checked	Operational Y/N?	Issues Found/action taken